




Micro10+



4X1

A Single product for all pre-sterilization steps

- 1 **C**leaning
 - Alkaline Detergents
 - Non-Ionic Surfactant
- 2 **D**isinfection
 - New Generation of QUATS
 - Broad Spectrum : (Bactericidal-Fungicidal-Virucidal)
- 3 **L**ubrication
 - Lubricates all Joints even the most Delicate ones
- 4 **P**rotection
 - Corrosion Inhibitors

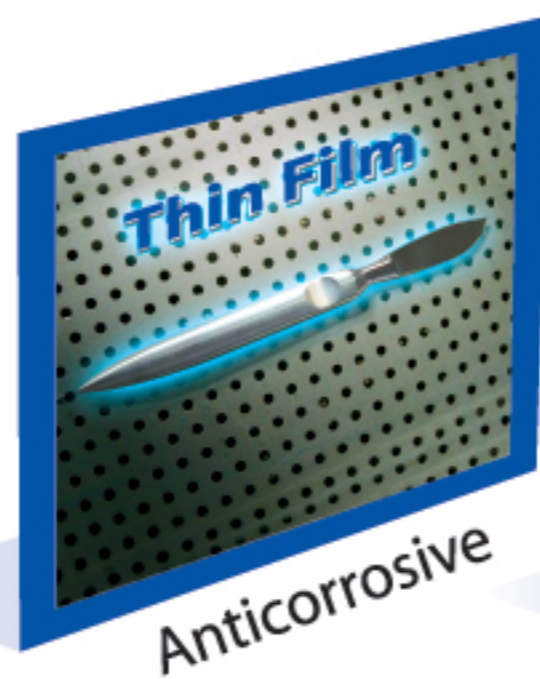



Instruments Disinfection

Micro10+

- The perfect solution for all surgical instruments whether used for soaking, or in an ultrasonic cleaner.
- *Micro10+* completely eliminates blood, saliva, debris & stains.
- *Micro10+* cleans, disinfects, protects & lubricates all your instruments even the most delicate ones in a single process.

Properties

- Aldehyde free.
- Optimal compatibility with all materials (Metal, Rubber, Plastic, Glass).
- Powerful cleaning.
- Corrosion inhibitors.
- Environmental friendly (Biodegradable 100%).
- Ensures efficiency and effectiveness.
- Odourless.
- Economic (1 L Conc. = 100 L Dil.).

Active ingredients

- Combination of new generation of (QUATS)
Alkyl benzyl dimethyl ammonium chloride .
- Exceptional cleaning agents
 - Alkaline detergents.
 - Non ionic surfactants.
- Corrosion inhibitors.

Spectrum

Broad spectrum, active against:

- Mycobacterium (T.B).
- Hydrophilic viruses (Poliovirus, Rotavirus, Adenovirus).
- Fungi (Candida albicans).
- Antibiotic resistant bacteria (MRSA -VRE).
- All vegetative bacteria (gr+ve & gr-ve).
- Lipophilic viruses (HBV-HCV-HIV).

Available in




Ref. : AFNOR, OFSP, DGHM, European Directive 93/42/EEC

Add : 10 Abo Sholoaa St, Off Sekket El-Zaher, Cairo, Egypt
Tel. : + (202) 59 31 459 - Fax: + (202) 59 31 467

Methods of use

Pre-Sterilization




1% for 15 minutes
(10ml / 1 L of tap water)

Soaking


Ultrasonic Cleaner

Washing Machine

Disinfection




2% for 60 minutes.
(20ml / 1 L of tap water).



5% for 15 minutes.
(50ml / 1 L of tap water).

Instruments Preparation Cycle

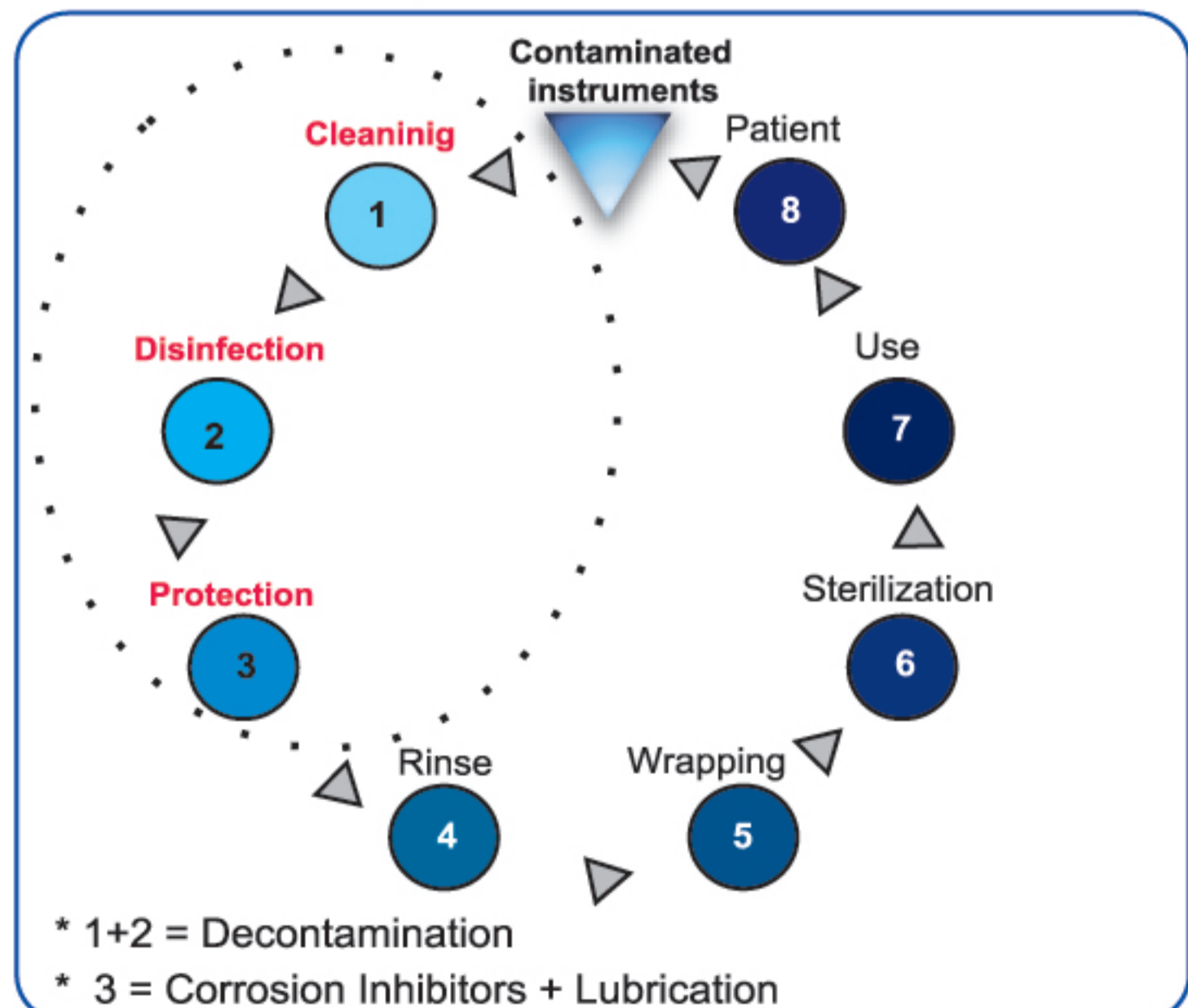

Website : www.abpharma.com.eg
E-mail : info@abpharma.com.eg